SHOWER STOOL

Art. 302070, 302071, 302072, 302073, 302078,
302079, 30207X

Gebrauchsanleitung
Brugervejledning
User Manual
Manuale
Bruksanvisning
Instrukcja Obsługi
Manuel de l'utilisateur

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mobilex.dk/medias/2021-05/ce_aqua_302070_73_302078_79_3020x7_7245.pdf

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1. Introduction

Thank you for buying a high-quality Mobilex product. These instructions of use contain a description and important information about the use of the medical device. It is intended to help you to use the product correctly and safely. It is very important to read the instructions carefully before using the device. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly trying to improve and upgrade its products. Therefore we reserve the right to change specifications and technical data of the product without further notice.

Intended use of the device

Shower stool is a medical device intended only to be used by patients that need support during washing under the shower. The device is suitable for persons who as a result of various diseases need a stable support to get easy access to all human body parts during washing. The device should be used inside bathrooms only.

Indications

The device is indicated for people with functional limitations, people in wheelchairs and other people with limited mobility of the lumbar spine, knees or hips, and for all those who have difficulties performing everyday personal hygiene in standing position. Any use other than stated above is prohibited.

Contraindications

The device is not indicated for those patients who can't maintain proper body balance in a sitting and standing position. The device must not be used if the user has impaired perception. Contraindications cannot be related to one specific clinical picture, and they depend on individual evaluation of potential user's health status. The choice of orthopedic aid appropriate for a given clinical picture must be always made with the help of a specialist physician or physiotherapist. In addition, observe the safety information in Chapter 2.

Areas of use

The product can be used in home care, health and long-term care facilities, and hospitals.

Medical incidents and unwanted events

In case of any unwanted events related to the device, please contact the dealer or manufacturer:

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 Email: info@mobilex.dk
---	--

2. Safety requirements

- Make sure that this manual has been read by all persons using the device.
- The manufacturer assumes no responsibility for damage and/or injury resulting from failure to follow the operating instructions.
- Only use the product in a flawless condition.
- If defects or errors are detected you must immediately stop using the device and contact the dealer.
- Follow all manufacturer's recommendations and warnings.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Don't climb onto the device.
- Avoid making constructive changes to the device, unless you have the manufacturer's written acceptance for such modifications.
- The device must not be loaded with more weight than indicated in Technical data.
- The device must only be used on stable surface.
- Use caution when assembling or adjusting any parts of the device.

Parts supplied

The device is supplied in a carton box. The contents of the package are:

- ✓ 1 Main frame;
- ✓ 4 Legs;
- ✓ 1 Seat;
- ✓ 1 User manual;

Preliminary check

Check the contents of the box with the list above. If at this stage you discover that a part is missing or damaged, contact your supplier immediately.

3. Description

Product description

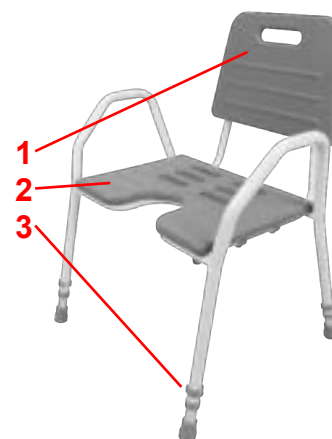
The shower stool is designed for individuals who require support during showering or while bathing. The frame (including legs finished with rubber feet) is made of aluminium tubing that makes it corrosion resistant and the seat is made of PU. No tools are required for assembly.

The device consists of the following elements:

1. Soft PU backrest (accessory)
2. Soft PU seat
3. Leg length adjustment clip

Assembly description

The shower stool is delivered in a cardboard box in which the lower parts of the legs are included separately. To assemble the shower stool, insert the 4 legs into the tubes and secure them at the desired setting using the clips provided (see **Adjusting the height of the shower stool**). Check if the soft PU seat is correctly mounted on the frame, as it may have come loose during transport. It may be necessary to push the seat down on to the frame while aligning the pins with the openings on the frame.

Assembly of the backrest (accessory)

The backrest of the seat can be mounted by removing the four locking clips under the seat, insert the the tubes from the backrest into the the corresponding openings in the tubes under the seat and reinsert the clips.

Function description

Adjusting the height of the shower stool

To adjust the height of the stool all legs must be adjusted to the desired length. To do so take the leg length adjustment clip out of the leg, slide the leg to the desired setting (in steps of 2,5 cm) and put the leg length adjusting clip back in the opening. Repeat for the other legs and make sure all four legs are adjusted to the same setting.

WARNING! Don't adjust the height of the shower stool when someone is sitting on it and always make sure all locking clips are mounted correctly.

NOTE: The seat height is correctly adjusted when the user is sitting on the shower stool and both feet fully touch the ground.

Accessories

The device can be equipped with the following accessories (not a part of the standard equipment):

- ✓ Soft PU backrest

4. Technical data

Shower stool	Small (302070+71)	Standard (302072+73)	Large (302078+79)	XL (30207X)
Dimensions (W x D x H)	51,5 x 53,5 x 41,7-58,5 cm	56,5 x 53,5 x 41,7-58,5 cm	62,5 x 53,5 x 41,7-58,5 cm	68,5 x 53,5 x 41,7-58,5 cm
Seat dimensions (W x D)	40,7 x 33,5 cm	45,2 x 33,5 cm	50 x 33,5 cm	50 x 33,5 cm
Recommended user heights	150-210 cm	150-210 cm	150-210 cm	150-210 cm
Width between armrests	42 cm	47 cm	52 cm	57 cm
Seat height (from the ground)	41,7 - 58,5 cm	41,7 - 58,5 cm	41,7 - 58,5 cm	41,7 - 58,5 cm
Armrest height	60,4 - 77,1 cm	60,4 - 77,1 cm	60,4 - 77,1 cm	60,4 - 77,1 cm
Weight	2,7 kg	2,9 kg	3,3 kg	3,86 kg
Max. user weight	150 kg	150 kg	200 kg	250 kg

Product label

Example of the product label

	QR-code link to homepage		CE marked product
	Manufacturer		Manufacturing date
	Observe the user manual		Max load
	Serial no.		Medical Device
	Item Number		Max width
	To be used indoor and outdoor		

Meaning of the icons

5. Care and maintenance

Regular dirt can be removed with standard cleaning products and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning. The entire product can be disinfected with a commercially available disinfectant. Always follow the disinfectant manufacturer's instructions for use and safety when disinfecting.

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following:

- ✓ Cleaning and disinfection of the product
- ✓ Checking the frame for bends, damage, wear or corrosion
- ✓ Checking wear parts and replacing them if necessary
- ✓ General re-tightening of the components

Recommended service life: Normally 10 years, depending on use.

Storage: We recommend storage in a dry place in temperatures above 0 °C.

Transportation

The device should be transported in a stabilized and secured way with no risk of sudden or dangerous movements during transport.

Reuse

The product is suitable for reuse. Before a new user takes over the product, the device must be checked on a technical level and disinfected according to point 5.

6. Most common problems and solutions

<i>Problems</i>	<i>Possible causes</i>	<i>Solutions</i>
Shower stool is not stable	Legs are adjusted to different heights	Adjust all legs to the same height

7. Disposal of the product

The product can not be disposed of with household waste and must be brought to the local recycling center.

8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

9. Declaration of conformity

The device has passed the conformity assessment in accordance with EU Regulation: MDR 2017/745 regarding medical devices and is CE marked. A declaration of conformity has been issued for the device and is available at:

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