## FOREARM CRUTCH

Art. 315021

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## 1. Introduction

Thank you for buying a high-quality Mobilex product. These instructions for use contain a description and important information about the use of the medical product. It is intended to help you to use the product correctly and safely. It is very important to read the instructions carefully before using the product. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly trying to improve and upgrade its products. Therefore we reserve the right to change specifications and technical data of the product without further notice.

### Intended use of the product

Forearm crutches are designed for use by people with limited mobility. They are used as an orthopaedic aid to support balance, walking, sitting and standing. The product is designed for indoor and outdoor use.

### Indications

The product is intended for the elderly, people with limited mobility of the lumbar region, knees or hips and for all those who have difficulties with standing and walking without an additional support point. The user is required to be able to balance independently while sitting and standing, as well as have adequate ability to move independently.

### Contraindications

Do not use the product if you have problems with balance or perception or if you have insufficient muscle strength to maintain a standing position. The choice of an orthopedic aid adapted to the patient's condition and needs must always be made by a medical specialist or physiotherapist.

### Areas of use

The product can be used in home care, health and long-term care facilities, and hospitals.

### Medical incidents and unwanted events

In case of any unwanted events related to the product, please contact the dealer or manufacturer:

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 Email: info@mobilex.dk
---	--

## 2. Safety requirements

- Make sure that this manual has been read by all persons using the product.
- The manufacturer assumes no responsibility for damage and/or injury resulting from failure to follow the operating instructions.
- The product should be completely assembled, adjusted in height and secured before use. Please contact us if there are any problems in this regard.
- Foldable and adjustable parts can become a cause of danger if a body part gets into the gap between the parts and gets trapped when the gap narrows.
- Always check that the product is rigid and well tightened before each use.
- Be careful on wet or slippery surfaces.
- Use the product only for the purpose intended by the manufacturer.
- Use extreme caution with the forearm crutch on stairs and slippery surfaces.
- Use the product only in good condition.
- Do not exceed the maximum load of the product.
- Clean the product regularly.
- Always check the stability of the forearm crutch before use.
- The rubber feet must not be excessively worn.

### Parts supplied

Each product package should contain: 1 forearm crutch, 1 user manual.

### Preliminary check

Check the contents of the box against the above list. If at this stage you discover that a part is missing or damaged, contact your supplier immediately.

## 3. Description

### Product description

The forearm crutch from Mobilex gives you a sense of security and fits very well in your hand thanks to the handle being made of soft-touch TPE. Since the forearm crutch is adjustable in length, it can be adjusted to the user's height.

### Function description

#### Length adjustment

To adjust the length of the forearm crutch, first loosen the retaining ring by turning it counterclockwise. Once the ring is loose, fully press the locking button and slide the lower tube so that the button pops into a hole corresponding to the desired length. Lastly re-tighten the retaining ring.

**WARNING!** After adjustment and before use, make sure the retaining ring is tightened so that the unit is stable and there is no unintended movement. Failure to comply with this warning may result in personal injury.

### Assembly

The assembly instructions are on the last page of this manual.

### Accessories

There are no accessories provided for this product.

## 4. Technical Data

Forearm Crutch	Item no. 315021
Length	93-122 cm
Weight	0,52 kg
Max. user weight	150 kg

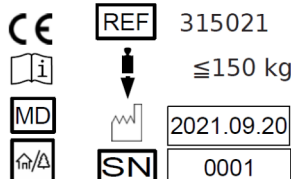
### Product label

MOBILEX A/S  
Grønlandsvej 5  
DK - 8660 Skanderborg  
Tel: +45 87 93 22 20  
www.mobilex.dk

### Forearm crutch

	QR-code link to homepage		Manufacturer
	CE marked product		Item Number
	Observe the user manual		Max load
	Medical Device		Manufacturing date
	To be used indoor and outdoor		Serial no

Example of product label

## 5. Care & Maintenance

Normal dirt can be removed with standard cleaning products and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (do not use alcohol-containing products and no solvents or abrasives).

The product basically does not need maintenance. Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following to be checked:

- ✓ Cleaning and disinfection of the product
- ✓ Frame/product should be checked if it has been bent, damaged, worn or corroded
- ✓ Wear parts are checked and replaced if necessary
- ✓ General re-tightening of the product

**Expected service life:** Normally 5 years depending on use.

**Storage:** We recommend storage in a dry place at a temperature above 0, away from intense sunlight.

### Transportation

The product is nomadic and easy to transport. During transport, the product should be secured in a way that eliminates the risk of unforeseen movements.

### Reuse

The product is suitable for reuse. Before a new user takes over the product, it must be checked according to point 5.

## 6. Most common problems and solutions

Problems	Possible causes	Solutions
When used the forearm crutch squeaks	The threaded bush is not tightened enough	Tighten the threaded bush

## 7. Disposal of the product

The product can not be disposed of with household waste but must be brought to the local recycling center.

## 8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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**Mobilex A/S**, Grønlandsvej 5, DK-8660 Skanderborg, Denmark, **SRN: DK-MF-000021885**