BISON walker
Art. 313300

Montage- und Gebrauchsanleitung
Montage- og brugervejledning
User Manual
Manuale d'uso e manutenzione

CE

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Indholdsfortegnelse

Table of content

Indice

	Montage-und Gebrauchsanleitung	3- 8	DEUTSCH
	Montage- og brugervejledning	9-12	DANSK
	User Manual	13-17	ENGLISH
	Manuale d'uso e manutenzione	18-22	ITALIANO
	Declaration of Conformity	24	DOC

1. Einleitung

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1. General information

This user manual contains a description of the product and important guidances to ensure a correct and safe usage of the product. It is important prior to use to read this manual carefully. It is especially important to read the safety requirements and follow these.

We continuously develop our products and we reserve the right without further notice to change the specifications and functions of products.

By questions please take contact to your dealer or directly to Mobilex A/S

Mobilex A/S	Phone:	+45 87 93 22 20
Grønlandsvej 5	Telefax:	+45 87 93 17 77
DK-8660 Skanderborg	E-Mail:	info@mobilex.dk
Denmark		

Intended usage

The walker has been designed as an assistance walking aid for people who have mobility problems related to standing or walking independently. With this device those persons can stand, and walk more easily. The main purpose of this product is to provide balance during the gait as well as providing partial unloading of the upper limbs through the forearm supports, which can be adjusted in and out to fit the users trunk. The forearm supports can be adjusted up and down by a control lever connected to the gas strut. The hand grip and brake lever assemblies can be positioned independently to enable the best position for the User. Such a walker offers better support, balance and stability than regular rollators. The device may be used indoor only. Any other usage is prohibited.

Indications

Individuals who, (because of the wide variety of possible health problems) require additional stable support during walking. Individuals under physical rehabilitation treatment after a surgical operation on their leg, foot or hip.

Contraindications

The device must not be used by persons who have serious problems with body balance or whose muscle strength is too low to be able to stand and walk, even with extra support of a walker. The need and possibility of usage of a walker by patient should be always estimated and indicated by a physician or physical therapist.

2. Security instructions

Make sure that this user manual is read by all persons who use the device. The manufacturer doesn't take any responsibility for damages and/or injuries, caused by the fact that the user's manual has not been followed.

- The allowed max. user weight for the device is 150 kg.
- The device is dedicated to be used only indoors on flat surfaces.
- The device may NOT be used as a stand-up helping aid.
- The gas spring is intended ONLY for support of adjusting the height of the armrests.
- The gas spring may not be used as additional power for standing up.
- The device may ONLY be used when the armrest supporting columns are locked in the right position with the position locking knobs.
- Use the device only for the purpose it is intended for.
- Use the device only in good technical condition.
- Avoid making constructive changes on the device, unless you have the manufacturer's written acceptance for such modifications.
- During different adjustments which are possible on the device the user must take care not to squeeze the fingers or other body parts between two moving elements of the device
- All wheels MUST be in contact with the floor at ALL TIMES during use. This will ensure the device is properly balanced and should avoid accidents.
- When using the device in a stationary position, the hand brakes MUST be locked.
- The brakes MUST be in the locked position before entering or leaving the device.
- The 4 wheels are respectively equipped with 2 total brakes and 2 directional brakes and hand brakes that can be easily activated.

3. Description

Product description

The Bison is a four wheel walker with a rigid frame made from powder coated aluminium with forearm supports equipped with push handles and brake handles for the user's safety and comfort. The device offers great manoeuvrability, stability and with forearm supports that can be adjusted vertically and horizontally. Bison is equipped with 4 parking brakes and 2 manoeuvre brakes activated by hand levers.

Delivery

Walkers are supplied in cartons, disassembled for easier transportation.

The contents of a package are:

- ✓ bottom frame with 4 wheels and brake handles
- ✓ 1 vertical supporting frame with 2 assembly bolts tighten in the bottom
- ✓ 1 set of forearm supports with 2 push handles
- ✓ 1 height adjustment gas spring with release lever
- ✓ 1 set of assembly tools
(10 mm hexagonal wrench, 6 mm hexagonal wrench, 13 mm standard wrench)
- ✓ 1 user manual

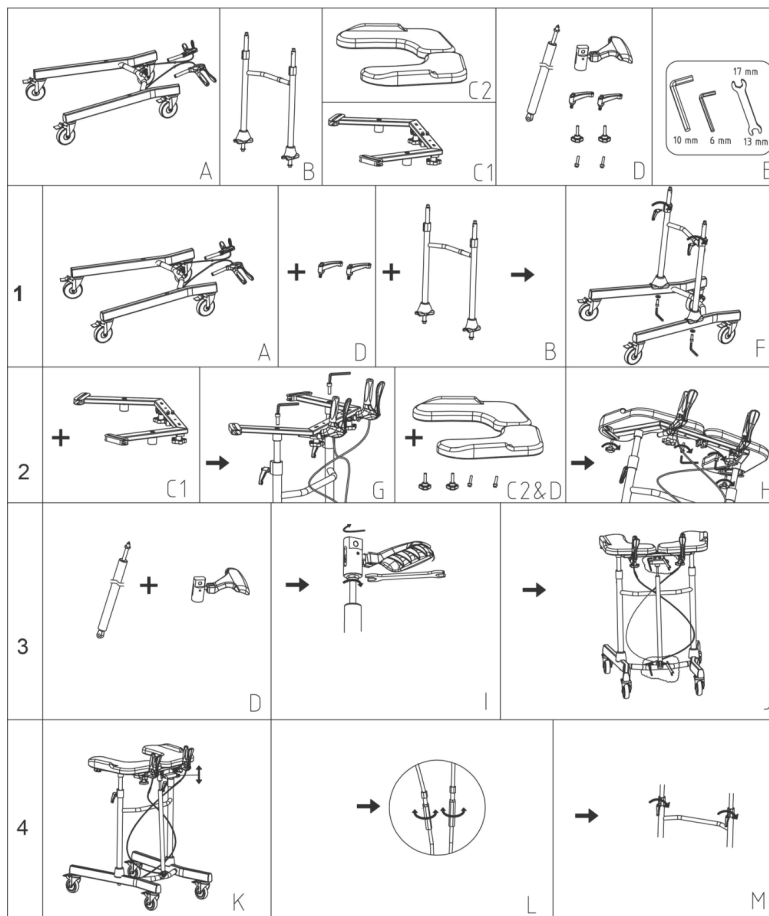
Initial Check

Check the contents of the box with the list above. If you discover at that stage that some part is missing or damaged, do not continue the assembly process. Contact your supplier right away.

Assembly

Please follow the assembly instruction steps shown on the back page of this manual.

Art no.: PLAT1000GB

Product details

 After assembly or adjustments and before usage make sure, that all locking knobs and levers are securely locked, so that the device is stable and solid. Assure there are no unintended movements. Failure to follow this warning can result in personal injury.

Adjusting the height of forearm supports

- Loosen 2 position locking knobs located on support columns;
- Unlock the red height adjustment lever by pulling it up,
- Make adjustment to your desired height by pushing the forearm supports down or letting them to move up freely.
- Release the red height adjustment lever
- Tighten the 2 locking knobs.

Correctly adjusted forearm supports should be located on the height of user's elbows when standing with arms directed downwards.

 **The device should only be used with the 2 locking knobs tightened**

Adjusting the width of forearm supports

Loosen the 2 position locking knobs located under the forearm supports;
Make adjustment of the supports to your desired width by sliding them outwards or inwards;
Tighten the 2 locking knobs.

Adjusting length and angle of the push handles

- Release the 2 position locking levers located under the push handles.
- Make the adjustment to your desired length and angle, to make it easy and comfortable to grab the handles.
- Tighten the levers.

Using the hand brakes

- TO STOP THE WALKER simply pull the brake handles rearwards towards the hand grips.
- TO REMAIN STATIONARY hold the brake handles in the "REAR" position.
- TO START MOVING, release the brake levers and parking brake by pushing the lever forward and the brakes will automatically release as you move forward.

Locking/unlocking the hand brakes

- To lock the brakes push the brake handles to the front until you can hear the "click" sound.
- To unlock the brakes pull the brake handles backwards until you can feel them release.

Locking/Unlocking the pedal parking brakes only front wheels

- To lock the brakes press with your foot on the brake pedal located at the wheel assembly until it locks. Repeat that for the other wheel.
- To unlock the brakes pull the brake pedal from the bottom with the nose of your foot until it unlocks. Repeat for the other wheel.

When using the device in a stationary position, the pedal brakes MUST always be locked.

You should always engage the brakes when you don't plan to move the device.

Never leave the device on slopes without brakes engaged!

Accessories

For the Bison walker various accessories are available with the following article no. 313380. Please contact your dealer.

4. Technical data

Bison walker	Item nr. 313300
Width of the frame at the rear wheels	66,0 cm
Width of arm rests min. - max.	66,0 - 80,0 cm
Distance between handles	34,0 cm
Height min. - max.	102,2 - 143,0 cm
Length	86,0 cm
Weight	18,3 kg
Max. load	150 kg

Product label

The product label is located on one of the bottom tubes of the main frame

Sample of the product label for Bison 313300

	QR-code link to homepage		Manufacturing date
	Manufacturer		Max load
	Observe the user manual		Max width
	To be used indoor and outdoor		serial no
	CE marked product		Item Number
	Max load basket		Medical Device
	Only to be used indoor		

Overview icon meaning

5. Maintenance and care

Normal dirt can be removed with standard cleaning agents and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (do not use alcohol-containing products and no solvents or abrasives).

The device basically does not need maintenance. Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following to be checked:

- ✓ Cleaning and disinfection of the product
- ✓ Frame/product should be checked if it has been bent, damaged, worn or corroded
- ✓ Wear parts are checked and replaced if necessary
- ✓ General retightening of the product

Recommended service life: Normally 10 years depending on use.

Cleaning the wheels

It is recommended to clean the wheels in hot water with a mild cleaning agent. Lubricate the bearings with oil periodically or when the wheels start squeaking.

Warning

 If the product is exposed to very calcareous water, a binding and accumulation of lime on wheel bearings and other moving parts may occur. In this case, the product can be cleaned with a limescale remover and then lubricated with lubricating oil.

Warning

 When used in highly chlorinated environments extra strong corrosion may occur and the warranty will be void.

Warning

 Please note that if the frame is bent or broken, the product must not be recycled but must be replaced

Storage

We recommend storage in a dry place.

Reuse

The product is suitable for reuse. Before reusing, where a new user is to take over the product, the product must be checked according to point 5.

6. General fixes

<i>Symptoms</i>	<i>Possible cause</i>	<i>What to do?</i>
The wheels don't turn smooth and easily	There is a dirt between wheels and forks	Clean the space between wheels and forks. Lubricate the axles of the wheels
Brakes aren't working correctly or one brake works stronger than the other.	The brake foot is torn	Replacement of brake foot or wheel required. Contact the service centre.
Gas spring cannot be released	Gas spring not adjusted correctly	Turn the height adjustment handle clockwise until it touches the release pin on the gas spring.

Transportation

The device is not a foldable product convenient for transportation. If it is necessary to transport it, we recommend to use a vehicle large enough, so there is no need to disassemble the device. If it is necessary to disassemble the device follow the assembly instruction from the last page, but in reverse order. The device should be transported and secured in a stable way and there is no risk of sudden and dangerous movements of the transported product.

7. Disposal of the product

The product can not be disposed of with household waste but must be brought to the local recycling centre.

8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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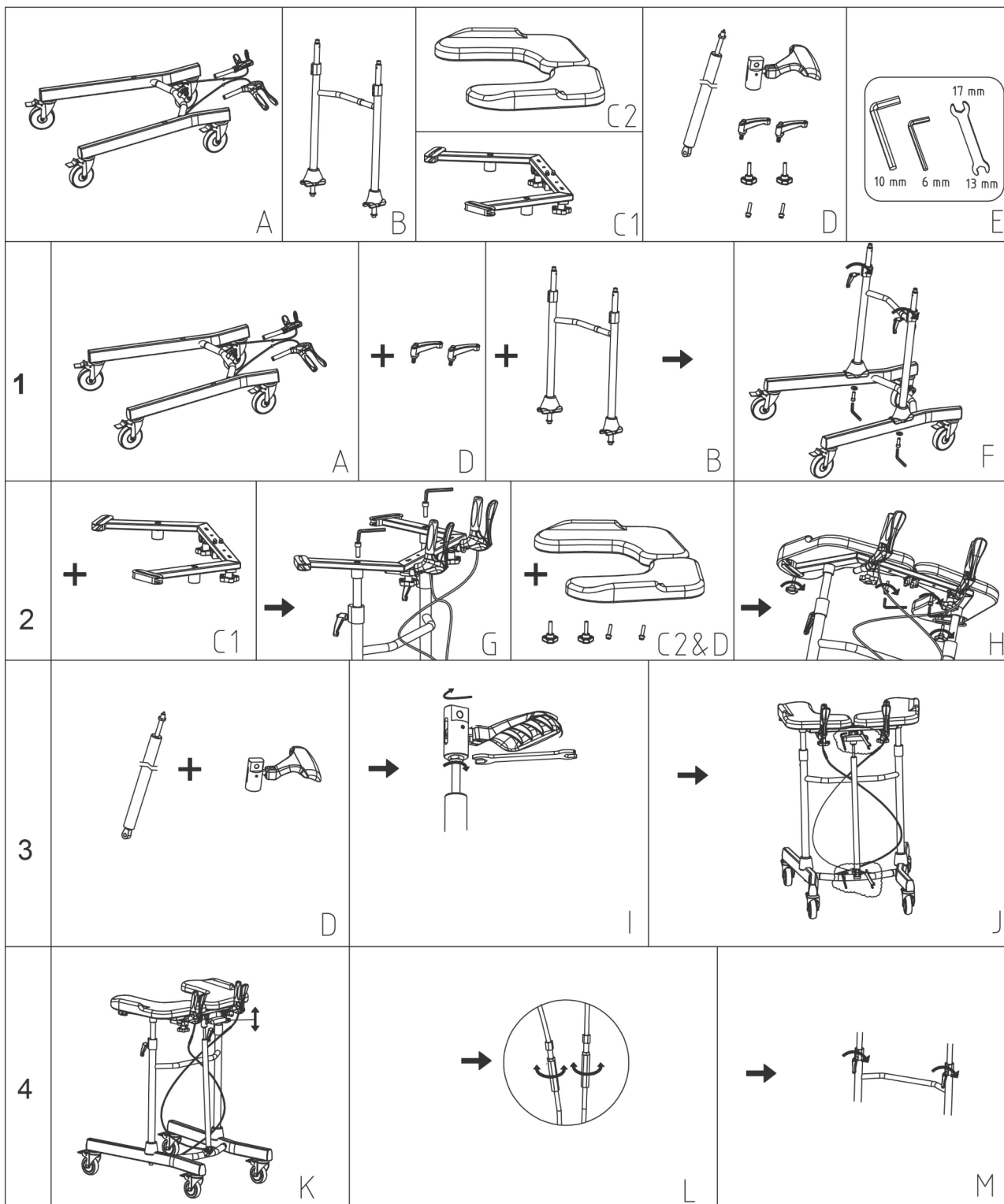
(Konformitätserklärung / Konformitetserklæring / Dichiarazione di conformità)

Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark,

Hereby declare under our sole responsibility as a legal manufacturer that the product specified on the product list below, meet the essential health and safety requirements and is in conformance with the provisions of the Regulation (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL on medical devices.

The product specified on the product list below is "technical aid for the disabled", classified as Class I, medical device. The classification is based on the requirements of Rule 1 of annex VIII, of the Regulation (EU) 2017/745.

The CE marking has been affixed on the product according to Annex V of the Regulation (EU) 2017/745.

PRODUCT LIST

Bison Walker - item no. 313300

ACCESSORIES LIST

Item nr.	Accessories item nr.
313300	313380

Harmonized norms used during conformity estimation:

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In case of unintended incidents or suspected faults, failures or defects of the product, the dealer or Mobilex A/S must be contacted /

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