Tekniske data

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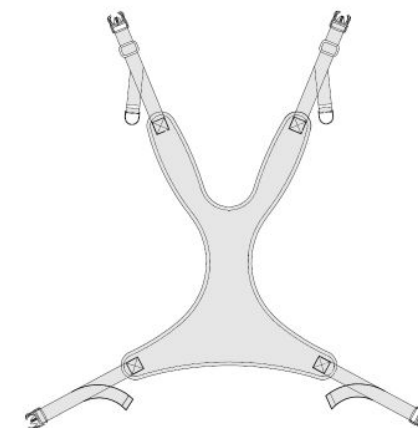
GB Universal front support belt

Technical Specifications

Item no.	Description	Material	Dimensions
233065	Universal front support belt	Polyester (PES), silicone foam	45 x 31 cm

Description

The front support belt is designed to restraint and prevent user in slipping forward when seated in a wheelchair or commode chair. The soft mesh fabric and silicone foam padding ensures very high breathability and comfort for the user. The butterflyshaped bodyform is especially suitable for womens upper bodyshape. The straps and hexagon shaped buckle closure device on the back allow for easy and correct installation on most commode chairs or wheelchairs.

Intended use

1. Position the user in the wheelchair or commode chair.
2. Place the support belt on the users chest and drape upper straps over users shoulders.
3. Make sure the shoulder straps fall strait down users back and chair backrest.
4. Pass the lower straps around user and behind buttom chair backrest.
5. Position the hexagon shaped buckle closure device on the chair backrest and close all buckles one by one.
6. Adjust the straps accordingly.

Safety instructions

- Make sure this IFU is thoroughly read before use.
- Product should be installed and fitted by a qualified healthcare profesional.
- Always check the label is readable and that the product is in good condition without signs of wear and tear.
- Only use product for intended purpose.
- Clean and maintain product regularly.
- In doubt, please contact the dealer who supplied the product.

Expected service life & end of life disposal

- Expected service life varies between 1-5 years, depending on condition of use and number of washing cycles.
- Expected service life is based upon a correct usage and washings procedure in accordance with this IFU and as stated on the product label.
- The support belt should be sorted as combustible waste.
- The packaging bag consist of soft plastic. It should be recycled according to local legislations.

Storing and shelf life

The support belt should be stored in a dry and clean environment. The shelf life is 3 years in original packaging. Store in temperatures between 0 - 40°C (32 - 104°F).

Product care & maintenance

The support belt is machine washable at 71°C. Always use a washing bag and remember to close the buckle prior to washing. Do not bleach, tumble dry, iron or dry clean.

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